# TSE 120

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## Scope of delivery

The scope of delivery includes:

- a raised toilet seat (body) including a rear support with fixing screws and knurled nuts, two front supports, two braces and two adjustable holders
- instructions for use
- 1 pair vinyl gloves

with Art. No. 520500400 also: two armrests

## Indication

Toilet aids allow or increase independence when using the toilet and performing the associated bodily hygiene activities, in particular by compensating for impaired or missing functions of the musculoskeletal system.

Toilet aids come into consideration for persons with a lack of trunk control and balance disorders, serious malfunctions of the lower limbs, stiff hips and spine and for those without hands, as well as quadriplegics or those with injured arms and significant grip malfunction.

## Application

The raised toilet seat TSE 120 is suitable for almost all standardized or standard toilet bowl's.

## Assembly

Remove the hinge and lid from the toilet bowl.

Assembly is carried out without tools. To do so, set the raised toilet seat on the toilet bowl, adjust both fixing screws M 8 after removing the knurled nuts to the appropriate width and insert through the holes of the toilet bowl.


Screw both knurled nuts loosely back onto the fixing screws, align the raised toilet seat, and then tighten both knurled nuts. Check all screws for tightness and tighten if necessary.


**Only the supplied fastening materials may be used for assembly.**

## For Art. No. 520500400:

Push the armrests sideways into the armrest holders provided for this purpose. When doing so, the arrows on the armrest and the armrest holder should match up.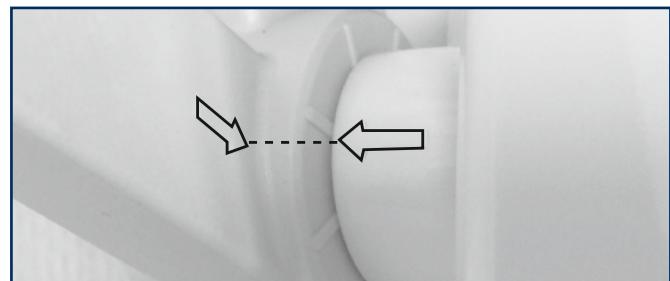

Now push the armrests in until they catch, and then fold them down so they rest on the support.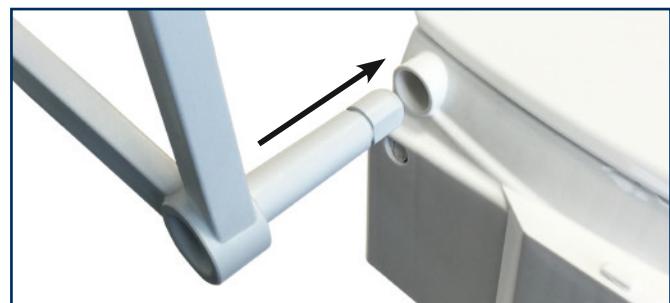


## Height adjustment



The raised toilet seat can be adjusted using the adjustable holders for an elevation of 8.5 cm, 12 cm or 15.5 cm. It is set at 12 cm at the time of delivery.

If you wish to change this, follow these steps:


- Lightly push the locking tabs on the right and left under the armrest supports and pull out on the body of the raised toilet seat.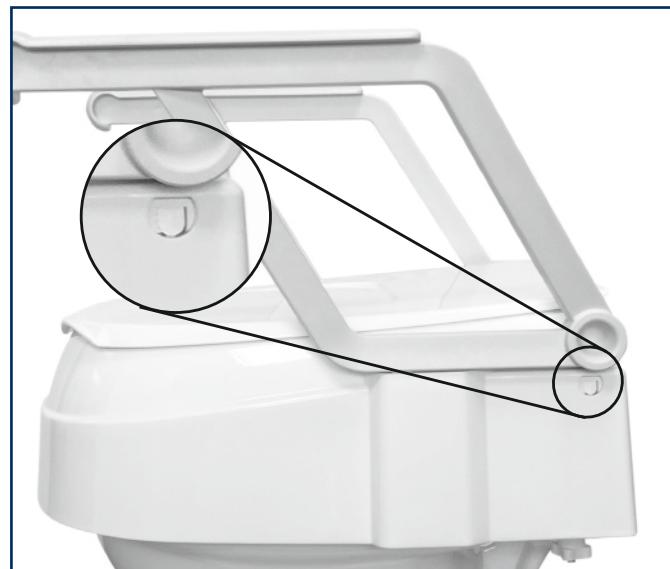

- Pull out the adjustable holders and set the desired height. To do this, insert the rear support into the corresponding hole in the adjustable holder.
- Take care that the adjustable holers/supports on the right and left are each connected at the same height.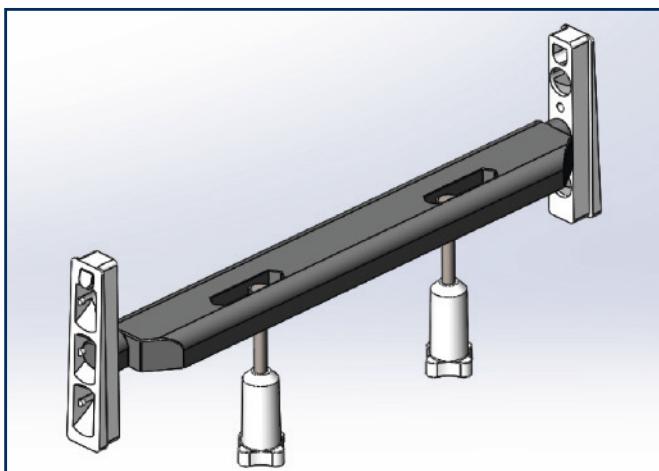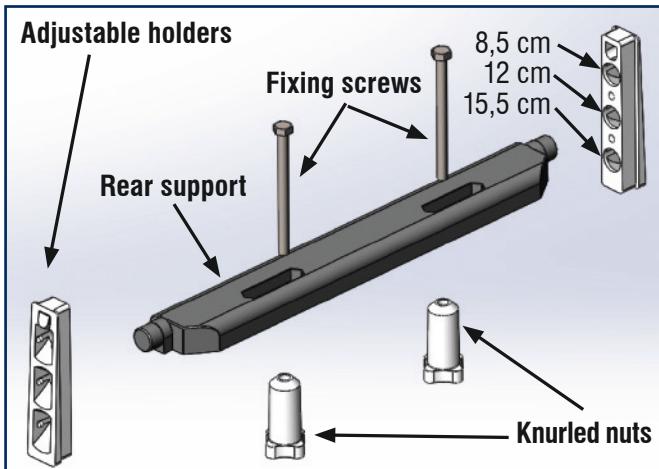

- To adjust the height in the front, insert the front supports (A) into the corresponding openings until you hear them click in place.
- Take care that the front supports on the right and left are each inserted at the same height.
- The front supports can be set lower than the rear supports.

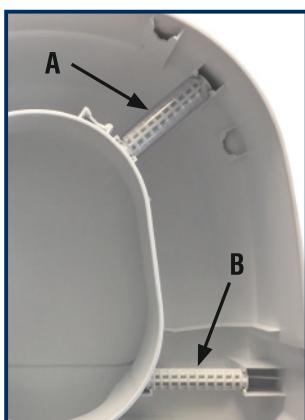- Insert the two braces (B) into the corresponding openings until you hear them click in place.
- Set the body of the raised toilet seat back into the rear support. In doing so, the holders of the rear support must engage with the locking tabs.

**The adjustable holders can only be inserted in one direction into the raised toilet seat.**

## Lid assembly

The lid is preassembled at the time of delivery, but can be replaced if necessary. To do this, turn the body of the raised toilet seat around and press the lid clips at the same time. The lid can then be taken out of the holders. Clip the new lid on.

## Safety Instructions

- The load on the raised toilet seat must not exceed 120 kg!
- The load on an individual armrest must not exceed 60 kg!
- The lid must not be used as a backrest in the opened position!
- The lid must not bear a load when closed (by sitting on it, for example)!
- Check the raised toilet seat regularly for a secure fit on the toilet bowl. Retighten the knurled nuts if necessary.

## Maintenance and Care

The raised toilet seat should be regulary cleaned with non abrasive cleaning materials.

## Warranty

The warranty period for the raised toilet seat supplied by us is 24 months.

Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

**Damage caused by improper use is not covered by the product liability.**

Our products are subject to regular revision.

## Reprocessing

The product is suitable for reprocessing in line with manufacturer's recommendations.

## Disposal

At the end of the product's lifetime, the components of the raised toilet seat should be disposed of properly. The parts are all recyclable and can be disposed of in the domestic refuse.

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Drive Medical GmbH &amp; Co. KG - D-88316 Isny/Allgäu (Germany)