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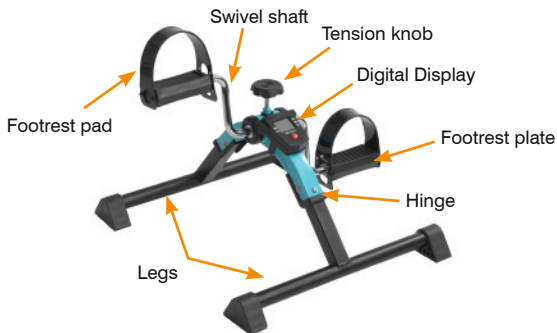
# assembly instruction

1. Remove contents from box.
2. Pull legs in opposite direction until locking button “snaps” into locking receiver. (see figure A).
3. Attach digital display to the peddler by sliding the loop over the tension knob receptacle located on the clasp on the back of the display attaches to the peddler frame. (see fig. B)

**Attention:** Only attach the multi-function display to this adhesive label!

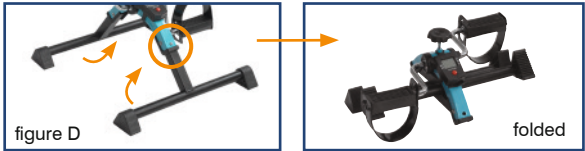
4. Insert tension knob on top of peddler and turn clockwise to tighten. The tension knob will increase/decrease the amount of tension on the peddler. Tighten/loosen to a level that feels of tension on the peddler. Tighten/loosen to a level that feels.

**Attention:** Do not stand on the pedal trainer! Do not put too much weight on the pedals, otherwise the pedal trainer may be damaged

## folding

To fold, depress push pin and fold leg inward. Repeat for second leg. (see figure D).

## automatic operation

1. **MODE (Red Button):** This control allows you to select a particular function.  
To Start, press and hold the red Button for 4 seconds. The LED will display the different functions (If after turning on and there is no activity, the unit will automatically shut off).
2. To select a function, wait until the 4 seconds set up is complete. Push the Red Button and the arrow will move through the functions. When the desired function is selected, simply use the peddler and the unit will calculate the exercise.
3. **COUNT (CNT):** Determines the amount of steps while operating the peddler.
4. **TIME:** Determines the amount of time the peddler has been operating.
5. **CALORIE (CAL):** Determines the amount of calories burned while using the peddler.
6. **REPS PER MINUTE (R.P.M.)** Automatically calculates the repetitions per minute.
7. **SCAN:** Will automatically review the results of all functions.

To reset the display to zero, press the Red Button (RESET) 3 seconds.

## battery replacement

If the display does not come on or reads incorrectly, the battery requires replacement.

To replace the battery slide the lid with retaining clip on the back of the display down, replace the underlying battery and set the display back on.

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## Two year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for two year of the original consumer purchaser.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to components, such as rubber accessories, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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Drive is a trademark of Medical Depot, Inc.

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© 2014 Medical Depot, Inc. Alle Rechte vorbehalten.