installation

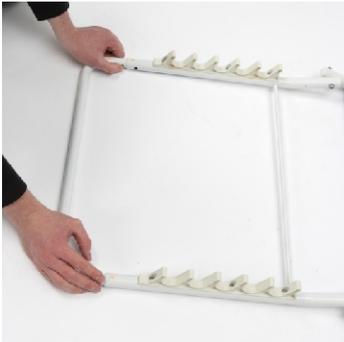
Step 1 / Schritt 1

Lay out main section. Insert the bar in to the two inner holes as shown.

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Step 2 / Schritt 2

Line up the bottom tubing so the YELLOW markers meet. Insert the tubing until the button clicks.

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Step 3 / Schritt 3

Line up the middle tubing so the BLACK markers meet. Insert the tubing until the button clicks.

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Step 4 / Schritt 4

Line up the top tubing so the RED markers meet. Insert the tubing until the button clicks.

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To adjust the angle:

Lift up the top tube of the backrest until the backrest is at the required angle. Position the cross bar in to one of the plastic grooves to secure.

To fold the backrest:

Lift up the top tube of the backrest, then remove the cross bar from the plastic groove and fold flat.

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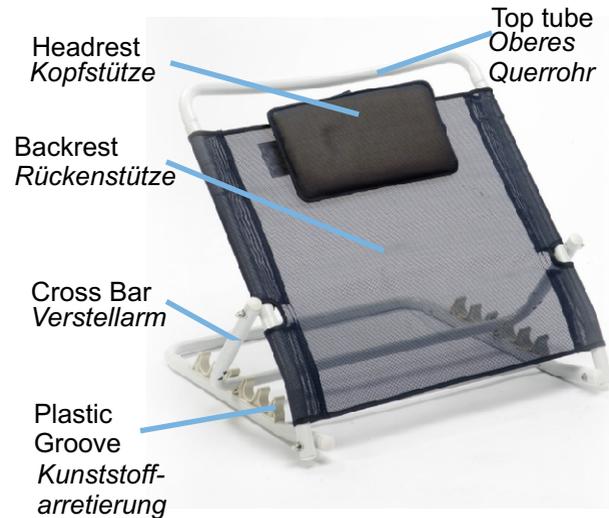
instructions

cleaning

The bed backrest can be cleaned using a damp cloth or mild detergent. Do not use abrasive or aggressive cleaning products as this could cause damage or irritation.

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warranty details

Your Drive brand product is warranted to be free of defects in materials and workmanship for two years from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

drive

Drive Medical Ltd

Ainley's Industrial Estate, Elland,
West Yorkshire, GB Hx5 9JP
www.drivemedical.co.uk

Doc code: BB001KDR_IFU (Iss 3/12)

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drive

www.drivemedical.co.uk